

ECOPALZER Medical Disinfector HE-3E/3PE



Designed to prevent nosocomial infection
Total disinfection is achieved with ECOPALZER!

The most remarkable disinfection chamber “ECOPALZER” in the world

HE-3E/HE-3PE

ECOPALZER Medical Disinfector



Anesthetic Machine Disinfection



Incubator Disinfection



Mattress Disinfection




D&D System

<Decomposition & Detoxification>


Disinfectant gas emission to the outside will cause environmental pollution. Needless to say; the strong odor and inhalation of residual formaldehyde-gas is harmful to health.

D&D System, developed with environmental preservation in mind, is radical system making detoxification and deodorization of -gas possible in a short time. **D&D System** has changed the future of formaldehyde-gas

D&D System

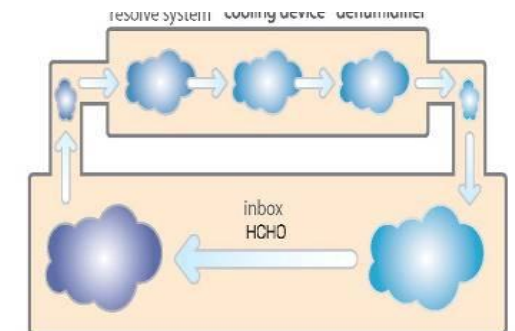

Chemical equation

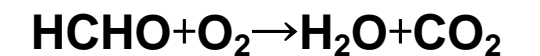

CHARACTER

- 1 Short Time Processing
One Operation 210~240min.
- 2 Low Running Cost
Pharmaceutical expense + Electricity Bill=AroundUS\$12
- 3 Harmless to Environment
- 4 Guarantee of Operator Safety
- 5 Easy Operation
By touching screen display according to purpose of use
- 6 Various Objects such as Mattress, beds and Anesthetic Machines.

Test Results


by Japan institute of Foods Ecology, Inc. Dec,20th 2006

Purpose Test of Bacterial Effect
Sample Incubator
"Bacillus Atrophaeus(ATCC9372)
ACE test Biological Indicator
Sterilization Formaldehyde gas
Condition 47°C whole process time 240min.
Use Model HE-3E


Testing Results

test	24H Later	48H Later	72H Later	Results
No.1	-	-	-	Perfect
No.2	-	-	-	Perfect
No.3	-	-	-	Perfect
No.4	-	-	-	Perfect
constant	+	+	+	imperfect

■HE-3E

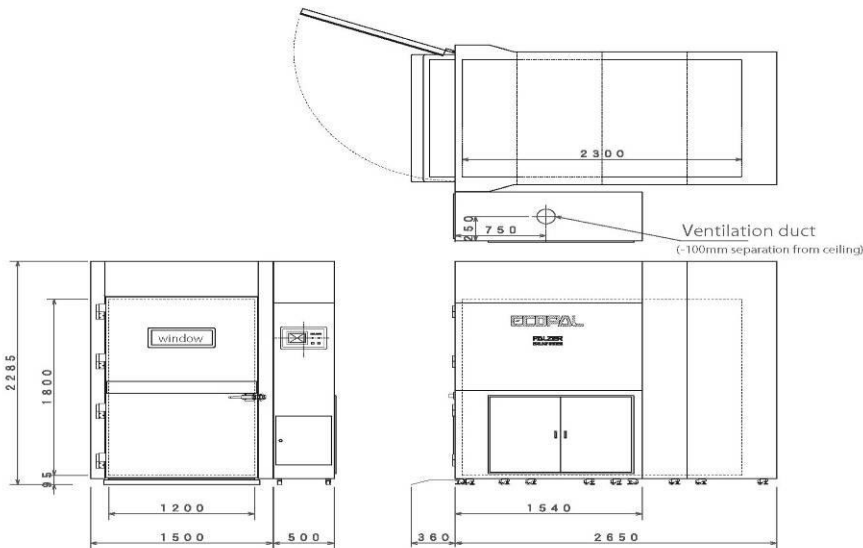

■HE-3PE

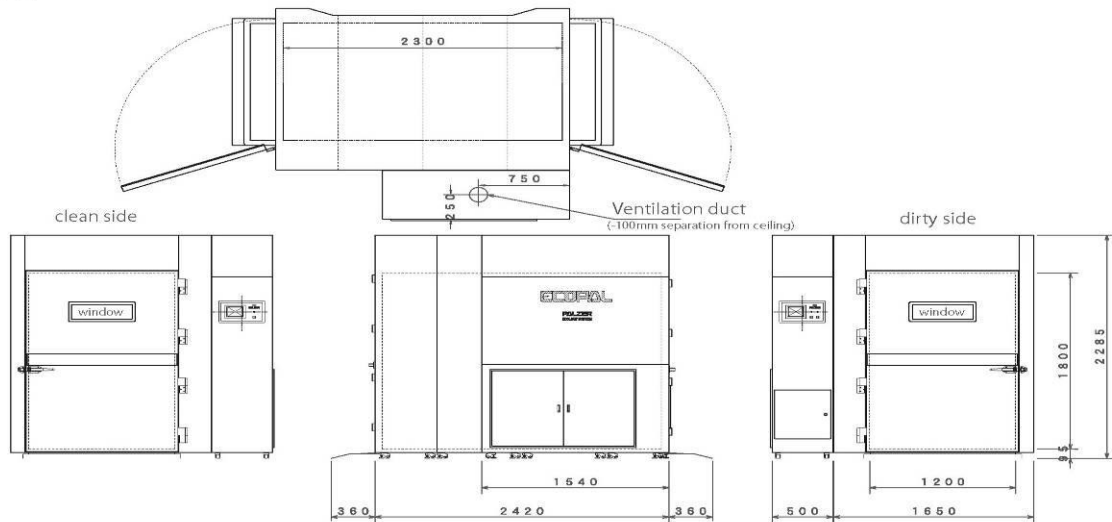

■HE Standard Equipment ECOPAL

Type	Size(wide×depth×hight)mm	Main voltage	objects
HE-3E	Out 2000×2650×2285 In 1200×2300×1800	Three-phase 200V/50A	Beds ,Mattress ,Incubator, Anesthesia apparatus , Other medical apparatus
HE-3PE	Out 2150×2400×2285 In 1200×2300×1800		



MEDIATE CO., . LTD

117-5 Megawa, Makishima-cho, Uji
Kyoto, 611-0041 Japan
Tel : +81-774-28-1170
Fax : +81-774-28-1169

Home page: <http://www.mediate-gp.co.jp>
E-mail: ecopal@mediate-gp.co.jp

Memo